




TUBE & DOME
measurement **GUIDE**



Measurement tool


Use the measurement tool to define the length. Be sure the red side is facing you when measuring the right ear measurements and the blue side is facing you when measuring the left ear.

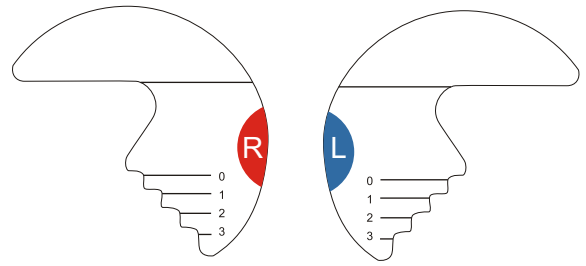

Measuring the ear


The measurement must be made horizontally. Place the measuring tool over the ear. It should fit firmly over the ear with little or not gaps.

Choose the size that most closely aligns with an imaginary horizontal line across the top of the ear canal (dotted line). In the example shown, the correct choice is size 1.

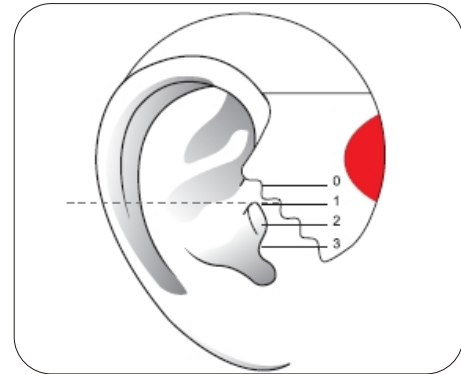

Selecting dome size

Check the ear canal and choose between small, medium or large dome. The dome should stay in the ear canal without moving out while being worn, talking or chewing.

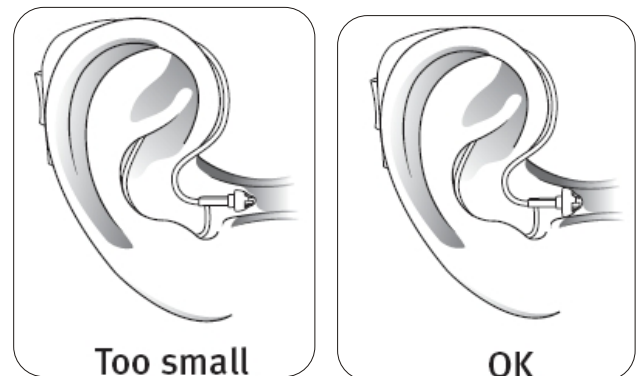

Print out measurement tool

It is important that this page is printed to 100% scale. To verify this, measure the printed distance between A and B with a US ruler. The distance should be no more and no less than 3.312". If you find that it is not 3.312", change the "Page Scaling" setting to 100% in the print dialog box and reprint this page .

